PACKAGING

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PHYSIOMER®

All the goodness of the sea to breathe better

Newborns and infants instinctively breathe through their nose, so it is important that their airways are clear. **PHYSIOMER® Baby Nasal Aspirator** gently relieves a baby's congested nose to help them breathe better in case of colds and rhinopharyngitis.

Its soft tip is adapted to the baby's nose and allows the excess mucus to be gently removed.

PHYSIOMER® Baby Nasal Aspirator for infants and young children*

Content: 1 Nasal Aspirator + 5 protective single-use filters

How to use PHYSIOMER® Baby Nasal Aspirator?

Before the first use rinse the device under hot water and dry it thoroughly. Proceed with nasal aspiration before feeding, when the child is awake and calm.

- 1- Wash your hands.
- 2- Insert a single-use filter in the back shell and close the device with the front shell.
- 3- Ensure that all components are properly assembled.
- 4- Lay your baby down on his/her back.
- 5- Place the soft tip of the nasal aspirator at the entrance of the baby's nostril and gently inhale through the mouthpiece. Mucus is stopped by the protective filter.
- 6- Repeat in the other nostril.
- 7- Gently lift the baby and wipe his/her nose with a tissue.
- 8- Throw away the used filter after each use.

After each use, rinse the **PHYSIOMER® Baby Nasal Aspirator** with hot water and dry it thoroughly. For hygiene reasons and to ensure proper performance, insert a new single-use filter before each use.

Use as often as necessary in case of nasal secretions.

For better removal of nasal secretions, use **PHYSIOMER® Unidoses** or **PHYSIOMER® Baby** spray prior nasal aspiration.

Refill pack of 20 protective filters sold separately.

Please refer to the leaflet of each product for optimal and safe usage.

* It is recommended to ask for medical advice/healthcare professional advice before using **PHYSIOMER® Baby Nasal Aspirator** for newborns or in case of birth nasal defects, recent nasal surgery or nasal injury.

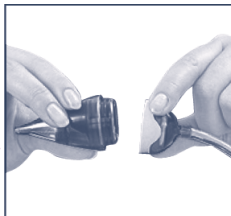
If symptoms persist for longer than 5 days, consult a healthcare professional.

Do not sterilise, do not boil.

Do not use after the expiry date.

Do not use the device if its original appearance has been altered or without a single-use filter.

Keep in its original packaging.

WARNING - CHOKING HAZARD – Small parts -

Keep out of reach and sight of babies and children.

Serious incidents must be reported to Laboratoire de la Mer and the local competent authority.

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Do not use a broken or damaged product

Discard the product in the bin

Recyclable paper

Medical device - Revision date: 2020/10