

PHYSIOMER®

PHYSIOMER® is a complete range of natural seawater nasal care products.

All the goodness of the sea to breathe better.

PHYSIOMER® Eucalyptus Nasal Spray

Decongestant - Cold Relief

Relief in 3 minutes only

For Adults & Children from 6 years old

135 ml

Properties

With *Eucalyptus globulus* essential oil

- Relieves nasal congestion due to cold and rhinosinusitis
- Helps combat infection by drainage effect
- Provides a pleasant sensation of relief and freshness

Natural Formula

- Hypertonic seawater with *Eucalyptus globulus* essential oil
- Preservative free

TO KNOW BEFORE USING PHYSIOMER® Eucalyptus

- Use by more than one person may spread infection.
- In case of nasal irritations, temporary tingling may appear during the use.
- Do not use after the expiration date printed on package.
- Do not use if the packaging has lost its original appearance.
- Do not use a broken or damaged product.
- If nasal symptoms prevail beyond 7 days for adults and children, consult a healthcare professional.
- Do not use when allergic to any of the components, in case of asthma, during pregnancy or while breastfeeding and for children under 6 years old.
- Keep out of reach and sight of children.

INGREDIENTS

97% hypertonic seawater solution (equivalent to 22 g/l of total mineral salts), sorbitol, natural «mint-eucalyptus essential oil» Flavour, Arlasilk PTM-LQ-(MV), E-Polylysine. Preservative free.

HOW TO USE

Recommended use

- For the relief of nasal congestion: 2 to 3 times a day, a 1 second spray ONLY in each nostril.

Administration mode

Adults and children: Adult supervision is recommended for use in children.

Before use: check if the spray works properly.

- 1 - Tilt the head to one side. Gently insert the nozzle in the upper nostril and press the nozzle once firmly for 1-2 seconds. Let the solution act for a few seconds.
- 2 - Repeat with the other nostril.
- 3 - Blow your nose.

Wash the nozzle with hot water after each use and dry properly.

